

The extra 60 seconds it takes to check the effectiveness of your glut solution prior to each use is vitally important to reducing the 2 million U.S. hospital-related infections that occur each year.*



How to Use MetriTest™ Test Strips

1. Remove one MetriTest Strip from container and replace cap immediately. (For MetriCide, use MetriTest 1.5% test strips. For MetriCide 28 or MetriCide Plus 30, use MetriTest 1.8% test strips.)
2. Dip indicator end of the strip into the MetriCide solution being tested for a full 2 seconds and remove immediately.
3. Lean strip against bottle (indicator end down) and wait 60 seconds. Use a paper towel underneath to absorb excess reagent dye.
4. Read color change at 60 seconds after strip is removed from solution. Disregard the top 2mm of indicator pad, which will be yellow. Pad will be completely purple to indicate the concentration of glutaraldehyde is above its minimum effective concentration (MEC). If ANY yellow appears, apart from the top line, the solution is below MEC and should be discarded. See visual standard below for color change interpretation.



* CDC: "CDC Releases New Hand-Hygiene Guidelines" Press Release by CDC Media Relations. October 25, 2002, <http://www.cdc.gov/od/oc/media/pressrel/r021025.htm>

Metrex is your "One-Stop Shop" solution in the ever changing world of infection prevention. We are here to partner with you to "Protect People" and provide a safe environment for all.

Protecting People


Metrex manufactures high-quality infection prevention products, including a complete line of enzymatic detergents, high-level disinfectants/sterilants, surface disinfectants, liquid medical waste disposal products, and hand hygiene products. Our customers include hospitals, outpatient clinics, emergency medical services, laboratories, veterinary clinics and educational facilities, to name a few. Our corporate headquarters is located in Orange, California, and our 85,000-square-foot state-of-the-art manufacturing facility is located in Romulus, Michigan.

Our mission is to "Protect People" by providing products that keep healthcare workers and their patients safe. Metrex, an affiliate of Sybron Dental Specialties, was founded in 1985. The Metrex products were brought together with the idea of improving infection prevention and providing one single source for all your infection prevention needs.

In addition, we provide education to the healthcare industry on how to use our products properly and how to get the best results using our products. Each Metrex sales representative is an infection control expert who provides education through presentations, articles, wall charts and videos. We have dedicated representatives throughout the United States and Canada.



1717 West Collins Avenue • Orange, CA 92867
(800) 841-1428 • FAX (714) 516-7904
www.metrex.com • e-mail: info@metrex.com

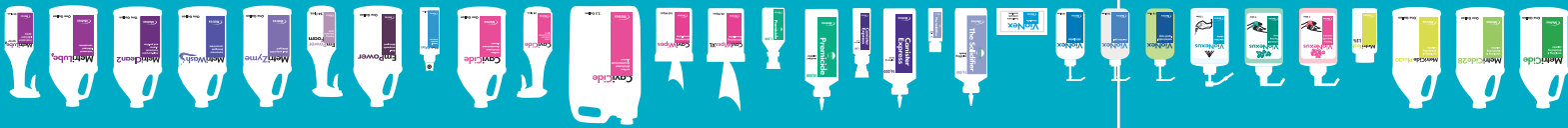

1717 West Collins Avenue, Orange, CA 92867 • Phone: (800) 841-1428 • Fax: (714) 516-7904 • www.metrex.com

©2005 Metrex Research Corporation



MetriCide®

Glutaraldehyde Testing Log Book

A Daily Record of MetriCide Test Results

Testing for Minimum Effective Concentration

For use with:

- MetriCide
- MetriCide 28
- MetriCide Plus 30

